



ABSOLUTE SOLIDIFIER R X

Instructions For Use

- 1. Wear hand and eye protection when handling infectious waste.**
- 2. Shake bottle vigorously to loosen settled material**
- 3. Before the procedure*, open accessory port on top of the suction canister lid.**
- 4. Empty bottle into suction canister or insert the dissolvable pouch via the accessory port. DO NOT CUT POUCH**
- 5. Close the accessory port.**
- 6. Place "Treated Liquid Medical Waste" Sticker on Canister**
- 7. Canister to be disposed in accordance with State, Federal and Local regulations.**

***Absolute Solidifier R X may also be added at the conclusion of the procedure**

***1 bottle or dissolvable pouch of the appropriate volume to 1 canister of the same volume**

SSP FORMULATION	SIZE: 500,1000,1200,1500,2000,3000	TOP: SPOUT/DP/TT
--------------------	---------------------------------------	---------------------

