



## **ABSOLUTE SOLIDIFIER**

### **Instructions For Use**

- 1. Wear hand and eye protection when handling infectious waste.**
- 2. Shake bottle vigorously to loosen settled material**
- 3. Before the procedure\*, open accessory port on top of the suction canister lid.**
- 4. Empty bottle into suction canister or insert the dissolvable pouch via the accessory port. DO NOT CUT POUCH.**
- 5. Close the accessory port.**
- 6. Canister to be disposed in accordance with State, Federal and Local regulations.**
  - Absolute Solidifier may also be added at the conclusion of the procedure**
  - 1 bottle or dissolvable pouch of the appropriate volume to 1 canister of the same volume**