



## Dust Covers

### Instructions for Use

- Dust cover is sealed using Rennco heat sealer or similar heat sealer designed to seal LLDPE film to file and paper to plastic peel pouches. Dust cover with a self-seal adhesive strip may also be used. Heat sealer temperature range between 170°C (225°F) and 110°C (230°F) will provide optimum seal.
- Load or control number and expiration label should be easily visible on the item placed inside the dust cover. Additional control number and expiration date label may be placed on the dust cover.
- Sterilized packs should be allowed to achieve room temperature prior to placing inside dust cover.
- While carrying dust covered item both hands should be used and placed on either end of the item with four fingers on the bottom using thumbs to stabilize the tray. Items inside a dust cover should not be carried holding the dust cover by the corner or one end. Proper handling of the dust cover should be done at all times to maintain the integrity of the dust cover.
- Opening dust cover should be done by grasping the dust cover with the thumb and fore finger at the notch and pulling in opposite directions.
- Once the dust cover is open contents can easily be removed.